


# Intect® 7

It only takes a minute to guarantee the integrity of your urine testing program.

## Test for the following Adulteration Parameters:

- Creatinine
- Nitrite
- Glutaraldehyde
- pH
- Specific Gravity
- Bleach
- Pyridinium Chlorochromate

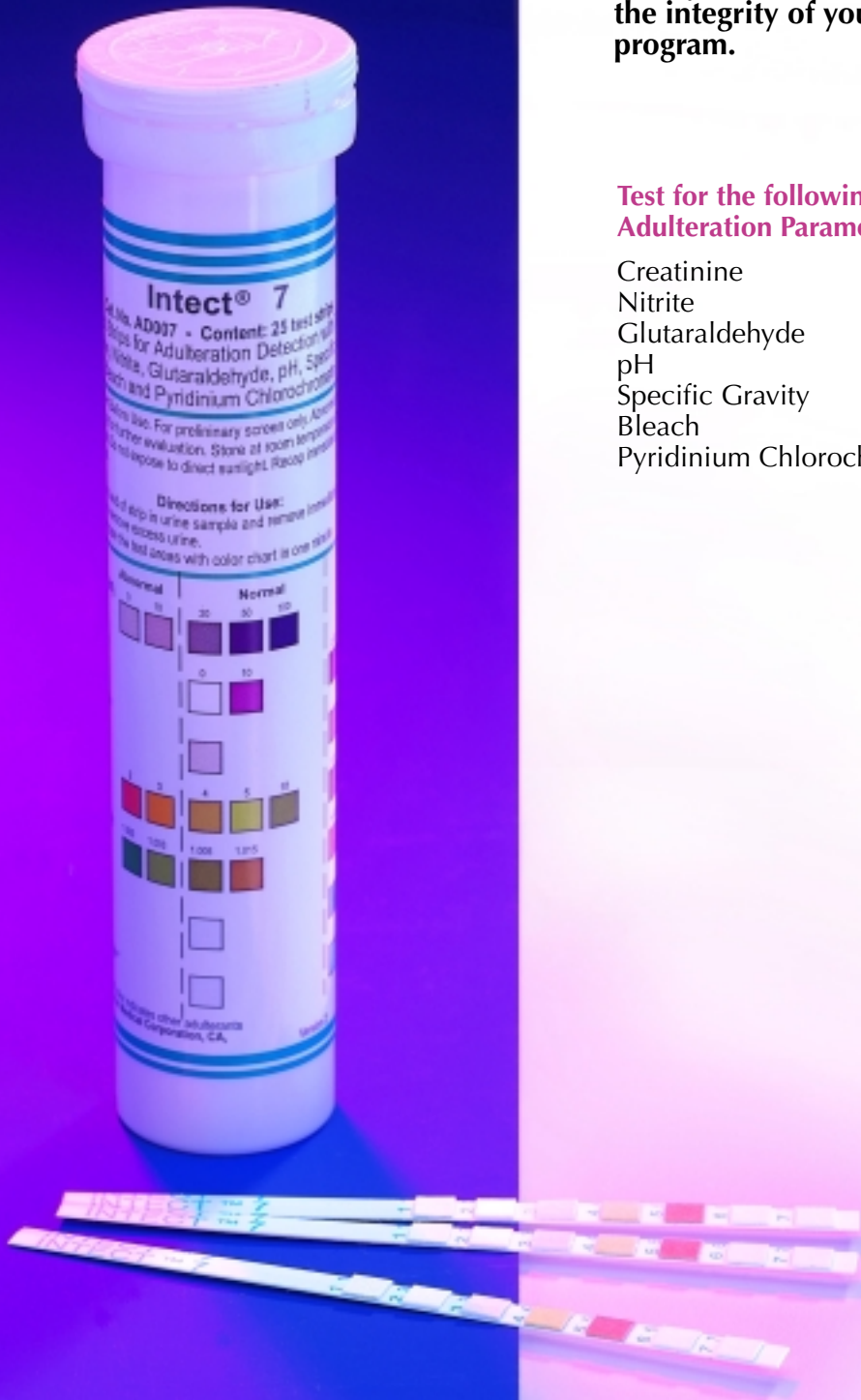

# Intect<sup>®</sup> 7 Urine Adulteration Test Strips\*

## The Facts

### THE FACTS

- Accurate, most comprehensive test available
- Detects adulteration, dilution and substitution
- Simple to use and interpret
- Results in 1 minute
- No instrumentation required
- Conforms to DOT guidelines
- 18 month shelf life
- IntectCheck<sup>™</sup> Controls available for monitoring performance

## Reactions

### TYPICAL ADULTERANT REACTIONS

	Creatinine	Nitrite	Glutaraldehyde	pH	Specific Gravity	Bleach	PCC

#### Classic Adulterants

	Creatinine	Nitrite	Glutaraldehyde	pH	Specific Gravity	Bleach	PCC
Bleach		•		•	•	•	•
Drano				•	•		
Vinegar					•		
Glutaraldehyde			•		•		
Sodium Nitrite		•			•	•	•

#### Commercial Adulterants

	Creatinine	Nitrite	Glutaraldehyde	pH	Specific Gravity	Bleach	PCC
PCC		•				•	•
Urine Luck 6.3							•
Stealth						•	•
Clean-X						•	•
Klear		•					
Instant Clean		•				•	•
Purafyzit						•	•
UR'n Kleen				•			


#### Dilution

	Creatinine	Nitrite	Glutaraldehyde	pH	Specific Gravity	Bleach	PCC
Urine Dilution	•				•		


## The Procedure

### THE PROCEDURE


- Dip the test strip into the urine collection cup and remove immediately. Be sure all pads are wet.

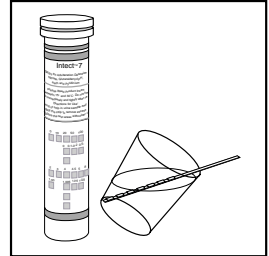

- Blot the test strip gently on its side to remove excess urine. Results are ready in one minute.

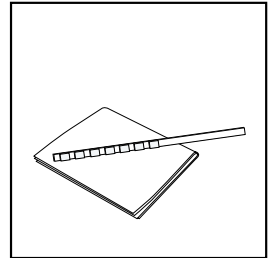

- Results Interpretation: Compare the color of the test with the color chart and interpret. Results are void after 2 minutes.

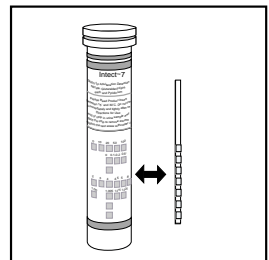

\*Patent Pending

For more information or to order contact:



Branan Medical Corporation  
10015 Muirlands Rd. Suite F.  
Irvine, CA 92618

www.brananmedical.com  
866-Intect7 (468-3287)