

# Drug Test Systems

8 Landing Way, Dover, NH 03820

Toll-Free: (888) 805-4849 • Fax: (603) 749-1118



---

## SCREENERS® SINGLE DIP PROCEDURE

---

Store the unopened tests at room temperature (59°-86°F). Check the expiration date on the foil pouch to make sure the test is still viable.

1. Pull the *protective cap* off the bottom of the test to reveal the absorbent testing strip. Do not discard the *protective cap*.
2. Carefully dip the testing strip into the urine sample. Do not allow the urine to touch the plastic casing of the test. Allow the test to absorb some urine - you will be able to see the liquid moving up through the window on the test. After five to ten seconds, remove the sample from the urine.
3. Replace the *protective cap* over the testing strip and lay the test on a clean flat surface while the test lines develop.
4. You can read negative test results (see below) as soon as they appear. Positive results must be read at least five minutes after removing the test from the urine sample. All results should be read and recorded before ten minutes have elapsed.
5. Check to see if there is a line in the *control zone* (marked with a **C**) of the test. If there is a line there, the test is functioning properly.
6. If there is ANY evidence of a line in the *test zone*, even the faintest hint of a line, that is a negative result. The *drug ID label* will tell you which drug the strip is testing for.

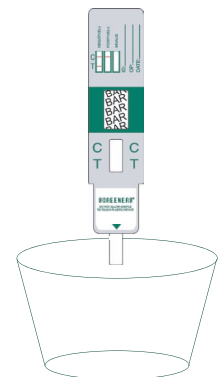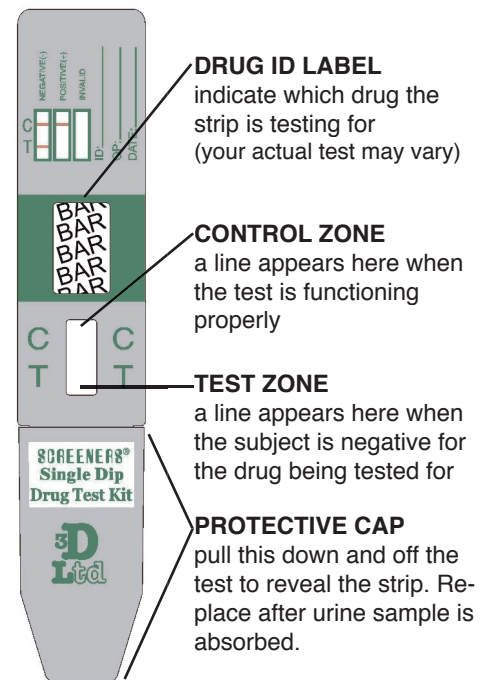


---

## READING THE RESULTS

---

### NEGATIVE RESULT:

When two bands appear, no matter how faint, one in the test region (T) and one in the control region (C). Any strip with both bands visible means a negative result.

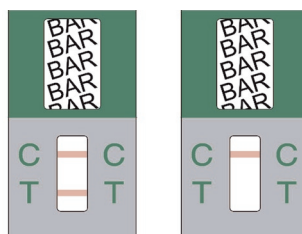

### POSITIVE RESULT:

When a band appears in the control region (C) and there is no trace of a band in the test region (T). Make sure the lines have a full five minutes to develop!